
# WOUNDSEAL®

## How to Apply WoundSeal Powder + Applicator:

Clean wound and apply only when bleeding resumes. WoundSeal Powder needs blood to work. For hard-to-reach wounds use enclosed applicator to apply powder.

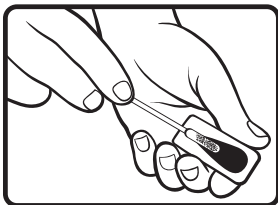### 1. Peel blister and load applicator.

### 2. Cover Wound completely with powder.

### 3. Press for 30 seconds using applicator or fingers, to form the protective scab and stop the bleeding.

**Repeat application if needed.**

The protective scab is safe to leave in place until it falls off naturally. Avoid wetting the scab for one hour. Occasionally upon application, a temporary warming or stinging sensation may be felt. If WoundSeal Powder must be removed as determined by a healthcare professional, brush away loose powder then soak wound in sterile water until scab starts loosening from wound. Gently scrape off loosened scab with gloved finger or tongue depressor.

#### **Warnings:**

**For external use only.** Not intended to replace stitches. Do not apply Powder to visibly infected wounds. May be harmful if inhaled or ingested. Avoid aspirating the product into the nasal or oral cavity. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Keep out of reach of children.

#### **Cautions:**

Store in a clean, dry place. Use by expiration date listed. Discard if product is damaged, opened, or if powder has clumped and does not flow freely. Discard contents after use since Powder is guaranteed to work only if it has not been left exposed to air or humidity.

One WoundSeal pack is sufficient to cover a wound area of up to 4.6 sq. cm (.7 sq. in.) to a depth of 2mm.

#### **Ingredients:**

Hydrophilic polymer and potassium ferrate.

Manufactured by Biolife LLC 8163 25th Ct. East. Sarasota, FL 34243.

WoundSeal is a registered trademark of Biolife, LLC.

Patents 6,521,265 and 6,187,347. Other US and International Patents Pending.

Made in the USA. All Rights Reserved. © 2014 Biolife, LLC.

For additional product information, call

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Fabricado por Biolife, LLC, 8163 25th Court E, FL 34243.
WoundSeal es una marca de comercio registradas de Biolife, LLC. Patente americana 6.521.265 y 6.187.347.
Fabricado en Estados Unidos. Todos los derechos registrados. © 2014 Biolife, LLC.