
WOUNDSEAL®


How to Apply WoundSeal Powder (for external wounds)

Clean wound and apply only when bleeding resumes.


WoundSeal Powder needs blood to work.

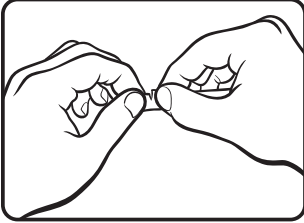1. **TEAR** package open at perforation.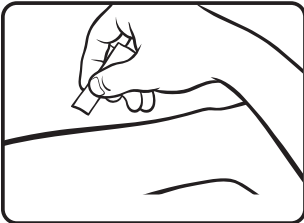

2. **POUR** onto bleeding wound, completely covering it.

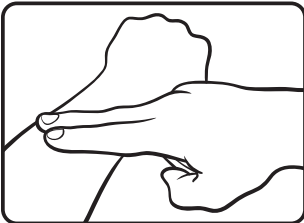3. **PRESS** for 30 seconds to form the protective scab and stop the bleeding.

Repeat application if needed. The protective scab is safe to leave in place until it falls off naturally. Avoid wetting the scab for one hour. Occasionally upon application, a temporary warming or stinging sensation may be felt.

If WoundSeal Powder must be removed as determined by a healthcare professional, brush away loose powder then soak wound in sterile water until scab starts loosening from wound. Gently scrape off loosened scab with gloved finger or tongue depressor.

Warnings: FOR EXTERNAL USE ONLY. Not intended to replace stitches. Seek medical attention if wound requires stitches. Do not apply WoundSeal Powder to visibly infected wounds. Keep out of reach of children. May be harmful if inhaled or ingested. Avoid aspirating the product into the nasal or oral cavity. Avoid contact with eyes; if contact should occur, thoroughly rinse eyes with water. Seek medical attention if any irritation persists or if bleeding continues as this could indicate a more serious problem.

CAUTION: Store in a clean, dry place. Use by the expiration date listed. Discard if dispenser is damaged, opened, or if powder has caked and does not flow freely. Discard contents after use, as WoundSeal Powder is guaranteed effective only if it has not been left exposed to air or humidity.

Ingredients: Hydrophilic polymer and potassium ferrate.

Manufactured by Biolife LLC 1235 Tallevast Rd. Sarasota, FL 34243.

WoundSeal is a registered trademark of Biolife, LLC.

Patent 6,187,347. Other US and International Patents Pending.

Made in the USA. All Rights Reserved. © 2010 Biolife, LLC.

For additional product information, call

1-800-722-7559,

or e-mail us at customer.care@biolife.com

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Fabricado por Biolife, LLC, 1235 Tallevast Rd. Sarasota, FL 34243.
© 2010 Biolife, LLC. Para información adicional de los productos, llamar 1-800-722-7559